

Wrappon · TREKKER WT-4 (S)

Infection Prevention & Risk Reduction



Minimizes the Risk of Secondary Infections

Why Choose Wrappon?

Reduces Workload

Automatic sealing process eliminates direct contact with waste.

Lowers Risk of Secondary Infections

Securely seals waste and viruses, ensuring a safer environment for both staff and patients.

Simple Disposal Process

Can be disposed of like regular diapers, including incineration when necessary.

Safe Containment of Contaminated Waste

Effectively seals masks, gloves, tissues, and other contaminated items.

Easy Cleaning & Maintenance

Seat can be disinfected with hypochlorous acid, alcohol wipes, or washed thoroughly.

Easy to use



Adding the special coagulant

After pouring the special coagulant, you finish the defecation. The special coagulant is to absorb the moisture from urine and human waste.



Pressing the remote control button

When finished, press the button on the remote control unit. It automatically rolls the deodorizing film and starts wrapping the urine and human waste. Then, the film is sealed and cut without being touched by hands.



Removing the wrapped film

The wrapping cycle is completed after 90 seconds when you hear the completed sounds as beep, beep. Then, just pull out the bottom tray and take it out.

Reliable Features of Wrappon

1

Special Odor-Blocking & Antibacterial Film – Prevents the leakage of odor and bacteria.



2

Dedicated Lithium-Ion Battery & DC Cable for Vehicles – Ensures continuous operation even in mobile settings.



3

Safety Handrails – Provides stability and ease of use for patients.



4

Compact & Space-Saving Design – Easily stored when not in use.



5

Safety Mechanism – Prevents accidental entrapment during heat-sealing.

Infection Control Measures

Proven Effectiveness in COVID-19 Wards!

During the global COVID-19 pandemic, toilets became one of the most critical facilities. With restrictions on shared toilets, many hospitals and care facilities adopted Wrappon portable toilets, which eliminate the need for bucket cleaning, providing superior hygiene and infection control.



In Disaster Situations

Power outages and water supply disruptions can render hospital toilets unusable!

Without functioning toilets, maintaining patient stability, accepting new patients, and ensuring the safety of medical staff become extremely difficult. Wrappon, which operates without water, has been deployed not only in evacuation shelters but also in numerous medical facilities during past disaster relief efforts.



Contact Information

 **NIHON SAFETY CO.,LTD.**

<https://wrappon.com/>

Wrappon Division, Nihon Safety Co., Ltd.
11F, Ichibancho Tokyu Bldg.
21 Ichibancho, Chiyoda-ku, Tokyo 102-0082 JAPAN
E-mail: wrappon.usa@nihonsafety.com



Wrappon Trekker
Medical

 is a registered trademark of NIHONSAFETY CO.,LTD.

These product descriptions are effective as of February 2023. For product improvement purposes, the specifications and external appearance may be changed without prior notice.